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EN: APPLICATION & USE: The earplugs provide protection against the harmful effects of noise. Do not insert the earplugs too deeply into the ear. It is recommended that you strictly follow all instructions of the manufacturer; otherwise the protective effect may be seriously impaired. Before use, check whether the earplugs are clean and undamaged. Place the earplug on the index finger and insert it fully by turning it slightly until the lamellae completely close the ear canal and are anatomically sound. The grip of the earplug should be facing inward towards the head when inserted into the ear. In noisy areas, the hearing-protection earplugs must be worn all the time. Furthermore, it is recommended not to modify the product and to exactly follow the instructions for use, storage and cleaning. The earplugs may only be inserted into the ears as intended for protection against noise and not into other body orifices. **BOLLSSEN - ERTG d.o.o.** is not liable for any damage arising from misuse. The product should be replaced in 01/10/2033 at the latest. **REMOVAL:** Pull the tab by slow-

ly rotating it. After removal, store the earplugs in the included aluminum container. **CLEANING:** When necessary, clean with a damp cloth or mild detergent (soap). Do not clean with chemicals or alcohol, as this can reduce the protective effect. **WARNING:** Sudden or quick removal of the earplug may cause damage to the eardrum. **IMPORTANT:** Store earplugs in case to keep clean. **NOTE:** If earplugs become cut, cracked or unpliable replace with a new earplug. **CAUTION:** Keep away from infants. This product is non-toxic but may interfere with breathing if caught in windpipe. For noise environments dominated by frequencies below 500 Hz, the C-weighted environmental noise level should be used. Improper fit of this device will reduce its effectiveness in attenuating noise. Consult the above instructions for proper fit. Although hearing protectors can be recommended for protection against the harmful effects of impulsive noise, the Noise Reduction Rating (NRR) is based on the attenuation of continuous noise and may not be an accurate indicator of the protection attainable against impulsive noise such as gunfire. The Noise Reduction Rating (NRR) calculated from the attenuation data is 17dB. **BOLLSSEN earplugs must be properly fitted to attenuate noise effectively. Refer to instructions. Tested according to ANSI Specs S3.19-1974, EN 353-3:303, CSA Z94.2-14 and AS/NZS 1270.** The level of noise entering a person's ear, when hearing protector is worn as directed, is closely approximated by the difference between the A-weighted environmental noise level and the NRR. **EXAMPLE:** 1. The environmental noise level as measured at the ear is 92 decibels dB(A). 2. The NRR is 17 decibels (dB). 3. The level of the noise entering the ear is approximately 75 dB(A).

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EN: Reusable earplugs based on the patented New Sound filter technology for optimal recovery, healthy sleep and noise-free nights. Used for sleeping, studying, reading, relaxing, protection against snoring. Environmentally Friendly - 100 Times reusable - Plastic free - Washable. Conforms to Directives / Regulation (EU) 2016/425. The EU Declaration of Conformity is available at: <https://bollsen.eu/ce>. Follow the instructions for use.

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17 DECIBELS
 (WHEN USED AS DIRECTED)

THE RANGE OF NOISE REDUCTION RATINGS FOR EXISTING HEARING PROTECTORS IS APPROXIMATELY 0 0 30 (HIGHER NUMBERS DENOTE GREATER EFFECTIVENESS)

BOLLSSEN - ERTG d.o.o. Ljubljana, Slovenia

BOLLSSEN New Sound

Federal law prohibits removal of this label prior to purchase

EPA LABEL REQUIRED BY U.S. EPA, REGULATION 40 CFR Part 211, Subpart B.

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Reusable earplugs based on the patented New Sound filter technology for optimal recovery, healthy sleep and noise-free nights. Used for sleeping, studying, reading, relaxing, protection against snoring. Environmentally Friendly - 100 Times reusable - Plastic free - Washable. Conforms to Directives / Regulation (EU) 2016/425. The EU Declaration of Conformity is available at: <https://bollsen.eu/ce>. Follow the instructions for use.

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